

How to set the program positions on the Swident Dental units. **Call 02 9567 355** for more information.

PATIENT POSITION 1

- a) Position the chair in the desired position.
- b) Lift the cover as indicated below and press the **1** button simultaneously. You will hear 3 short beeps which means the program is set.

PATIENT POSITION 2

- a) Position the chair in the desired position.
- c) Lift the cover as indicated below and press the **2** button simultaneously. You will hear 3 short beeps which means the program is set.

PATIENT POSITION 3

EXIT position. This is pre-programmed from the factory

PATIENT POSITION 4

LP (Rinse only). This is pre-programmed from the factory

. Please note that the chair should not be allowed to go down all the way to the bottom.

