PARTNER

Dental unit type:
Riunito modello:
Serial number:
Matricola:

▲ Prerequisiti • Pre-requirement	2
Pré-conditions • Prerrequisitos • Vorbedingungen	3
▲ Predisposizioni • Predispositions	6
Prédispositions • Preinstalaciones • Vorbedingungen	6
▲ Caratteristiche tecniche • Technical characteristics	7
Caractéristiques techniques • Características técnicas • Technische Merkmale	7
▲ Rotazioni possibili • Possible spins	7
Rotations possibles • Rotacions possibles • Mögliche Rotationen	7
▲ Installazione • Installation	8
Installation • Instalación • Installation	9
▲ Collegamenti elettrici • Electrical connections	14
Raccordements électriques • Conexiones eléctricas • Elektrische Anschlüsse	15
▲ Collegamenti elettrici base poltrona • Electrical connections to chair's base	14
Raccordements électriques à la base du fauteuil • Conexiones eléctricas en la base del sillón • Elektrische Anschlüsse am Sockel des Stuhls	15
▲ Regolazioni braccio articolato tavoletta • Regulations of the articulated arm of dentist's instrument table	16
Reglement de l'articulation du bras de la table d'instruments dentiste • Regulaciones del brazo articulado de la mesa • Einstellungen des Tischgelenkarms	17
▲ Regolazione tempo riempimento bicchiere e lavaggio bacinella • Time regulation of cup filling and bowl cuspidor flush	18
Temporisation de temps du remplissage du verre et du rinçage du crachoir • Regulación del tiempo de llenado del vaso y lavado de la escupidera	19
▲ Einstellung der Füllgeschwindigkeit von Gläsern und Mundspülbecken	19
▲ Sistema di gestione spray • Spray management system	18
Système de gestion des sprays • Sistema de gestión de los sprays • Anwendung der Sprühfunktion	19
▲ Impostazioni modalità operativa serbatoio liquido alimentazione strumenti	
Formulation operating modality liquid bottle of feeling instruments	18
Position opérationnelle du réservoir pour eau distillé aux instruments	
Regulación del modo operativo depósito líquido de alimentación instrumentos	
Einstellungen der Arbeitsweise des Wassersammeltanks für die Versorgung der Instrumente	19
▲ Dispositivi di regolazione planarità • Levelling of the device	20
Ajustements de la mise à niveau au plancher • Dispositivos de regulación de la horizontalidad • Vorrichtungen zur Einstellung der Ebenheit	21
▲ Planarità longitudinale • Longitudinal levelling	20
Mise à niveau longitudinale • Horizontalidad longitudinal • Ebenheit in der Länge	21
▲ Planarità trasversale • Trasversal levelling	20
Mise à niveau transversale • Horizontalidad transversal • Ebenheit Quere	21
▲ Planarità tavoletta • Levelling of the instruments table	20
Mise à niveau de la table d'instruments dentiste • Horizontalidad de la mesa • Ebenheit des Tisches	21
▲ Frizioni delle rotazioni • Clutches of the spins	22
Friction des rotations • Fricciones de las rotaciones • Reibungen beim Drehen	23
▲ Rotazione tavoletta • Spin of the instrument's table	22
Rotation de la table d'instruments dentiste • Rotación de la mesa • Drehen des Tisches	23
▲ Rotazione braccio articolato • Spin of the articulated arm	22
Rotation du bras articulé • Rotación del brazo articulado • Drehen des Gelenkarms	23
▲ Rotazione primo braccio • Spin of the first arm	22
Rotation du premier bras • Rotación del primer brazo • Reibung des ersten Arms	23
▲ Messaggi di errore • Error messages	22
Messages d'erreur • Mensajes de error • Fehleranzeigen	23

Prerequisiti

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Pre-requirement

The equipment that you will install is catalogued Class 1 Type B equipment.

The equipment is conformed EEC directive 93/42/CEE "Medical Devices".

Classification (according to the standard EN60 601-1):

- 5.3 Common Equipments (Equipment with a cover NOT protected against the water penetration);
- 5.5 The Equipments (units) have not to operate in areas with risk of explosion; and close to any apparatus producing electromagnetic emissions;
- 5.6 Equipments with continuous operation with intermittent load (=DAB).

Earth of protection

Attention, see enclosed documentation

Disposal conforming to the RAEE standard

Functioning:
230 V / 50 Hz / 320 VA
Operation way:
continuous with intermittent charge
The maximum capacity of lifting:
135 Kg. (except unit and lamp)

For a correct and sure installation of the equipment respect the following prescriptions:

- The floor should be conformed to the directive DIN 1055 B 1.3 and DIN 18560 T 1;
- A bipolar external switch must be installed, conformed to the rules with the following electrical characteristics: 250 V / 10 A, for disconnecting the equipment on internal replacement;
- The electric system must be conformed to the directive CEI 64.4, made by different conductors (with indices <30mA) to 250V/16A;

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Clutches of the spins

Spin of the instruments table

For regulating the clutch of the spin of the instruments table, disassemble the cover (1), unscrewing the screw (2), screw in a screw (3) for obtaining the appropriate clutch and tightening the blocking nut (4).

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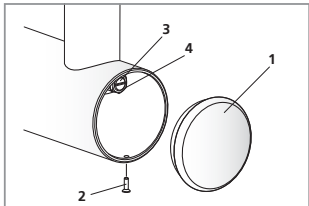
Fig. 26 • Pct. 26 • Abb. 26

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Spin of the articulated arm

For regulating the clutch of articulated arm's spin, disassemble the tap (1), unscrewing the screw (2), screw in a screw (3) for obtaining the appropriate clutch and then tightening the blocking nut (4).

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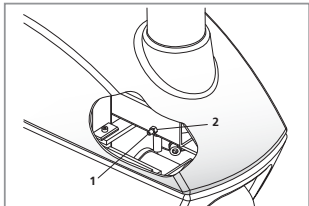
Fig. 27 • Pct. 27 • Abb. 27

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Spin of the first arm

For regulating the spin of the clutch of the first instruments table's arm, tighten the screw (1) for obtaining the appropriate clutch and tighten the blocking nut (2).

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Error messages

In case of disfunction, the unit signals to the technician the area where an intervention is required in order to restore the correct unit functionality. If a led signalling the instrument power (placed on the keyboard of the Dentist's control panel) is blinking, the device reports a malfunction in following areas:

- 1%: Set-up error of dip-switches in the control panel, concerning the instruments
- 25%: Dialog error between serial ports of the cards (control panel - chair)
- 50%: Foot-control error
- 75%: Set-up of the spray management mode.

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- Multiple moving socket or extended cables must be not connected to the system;
- Do not connect the specified devices not making part of the equipment;
- Particularly the grounding must be conformed to the art. 3.2.01 of the named above regulations.

The water pressure to the entrance of the unit must be between 1.8 and 3.0 bar. The line must supply a water volume at least 5 l/minutes.

The air pressure given from the compressor must be between 5.0 and 7.0 bar. The compressor must be the dry type by a value of given air 60lit/min by 5 bar.

It is recommended to install 2 taps of the air and water feeding lines.

The aspiration tube must be of \varnothing 32 - 40 mm with attack \varnothing 25 mm.

The predispositions must be done according to the indications in the following page and in the plan of installation in 1:1 as attached with each unit.

The fixing to the earth of the unit must be executed with dowels of minimum diameter of 10mm.

The manufacturer doesn't undertake the responsibility for safety properties of the equipment if:

- Installation, maintenance, repair and modifications are not carried out by a person explicitly authorized by us;
- The general and electric installation of the room is not conformed to the related laws;
- The equipment has not been correctly handled according to our "operating instructions";
- The equipment has not been correctly used according to our "User Manual".

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Levelling of the device

Longitudinal levelling

Loosening the fixing screws between the installation arm and the chair (1) and between the installation arm and water unit (2), it could be oriented to obtain the perfect longitudinal levelling.

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Trasversal levelling to the floor

Near to the 3 fixing screws (1) between the installation arm and the water unit, 3 push screws (2) are placed, through them it could be possible to obtain a perfect levelling.

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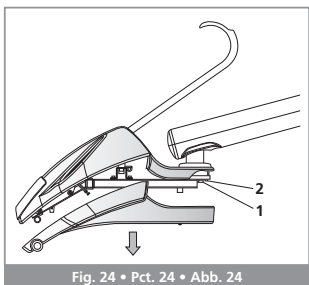
Fig. 24 • Pct. 24 • Abb. 24

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Levelling of the instruments table

Remove the inferior cover of the dentist's instrument table; near to the 3 fixing screws (1) of the table, 3 push screws (2) are placed, through them it could be possible to obtain a perfect levelling.

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Time regulation of cup filling and bowl cuspidor flush

The times of cup filling and bowl flush are regulated through timer placed on the electronic board of the patient chair. Through the timer (1) the time of cup filling (glass) is regulated. Through the timer (2) the time of the bowl flushing (basin) is regulated. Rotating the screw in clockwise sense, the time decreases while rotating the screw in anticlockwise sense, the time increases.

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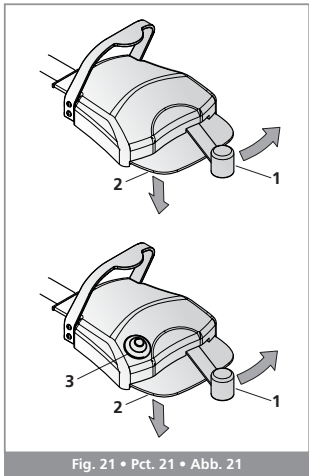
Fig. 21 • Pct. 21 • Abb. 21

Spray management system

Activation of the instrument spray can be configured using two different systems.

STANDARD SYSTEM: Turn lever 1 to turn the instrument on; press lever 2 simultaneously to activate the spray. If lever 2 alone is turned the chip is activated. No command or indication appears on the instruments table.

ALTERNATIVE SYSTEM: The spray must be selected from the instruments table using the "spray tank" button, which has a dual function:

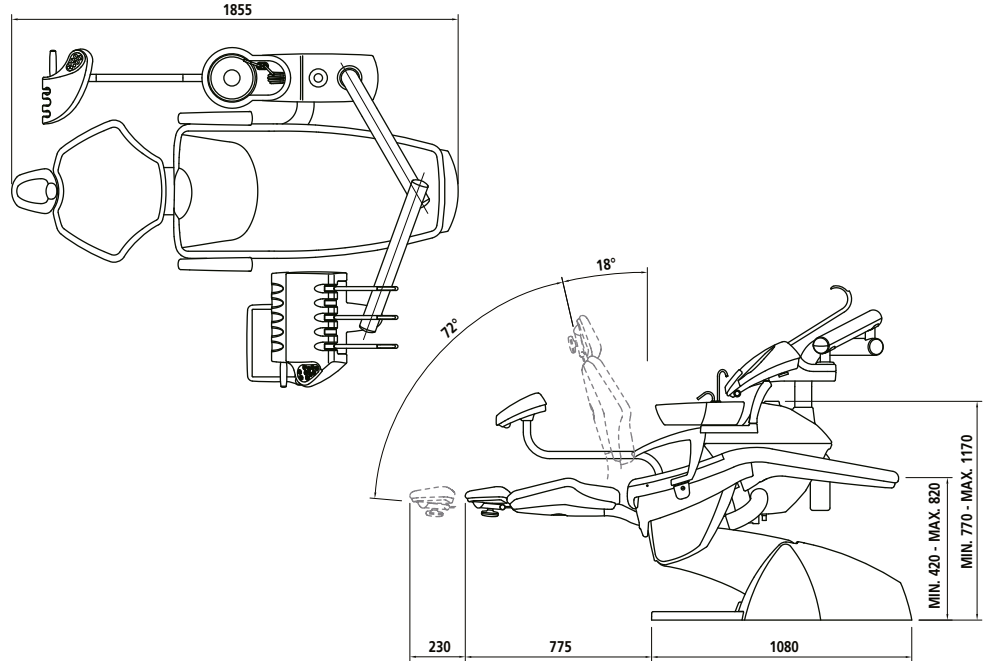
- With the instruments at rest the button controls and, when the LED is on, indicates insertion of sprays from the bottle.
- With the instrument raised the button controls and, when the LED is on, indicates insertion of the spray in the raised instrument.

The spray will be activated when lever 1 on the pedal is turned to start the instrument. If lever 2 is turned the chip is activated. Each instrument stores its last setting.


Selecting systems

- 1) Turn the water unit off.
- 2) Switch DIP1, DIP2 and DIP3 on the instruments board to the ON position.
- 3) Turn the water unit on; the 75% LED will flash continuously and the water unit will be unable to work normally.
- 4) Use the "BOTTLE" button to select the spray system:
 - The Standard System is set when the "Bottle" LED is off;
 - The Alternative System is set when the "Bottle" LED is on.
- 5) Turn the water unit off; the spray system has now been stored.
- 6) Return DIP1, DIP2 and DIP3 to their correct positions.
- 7) Turn the water unit on. It can now be used normally.

Caratteristiche tecniche • Technical characteristics
Caractéristiques techniques • Características técnicas • Technische Merkmale

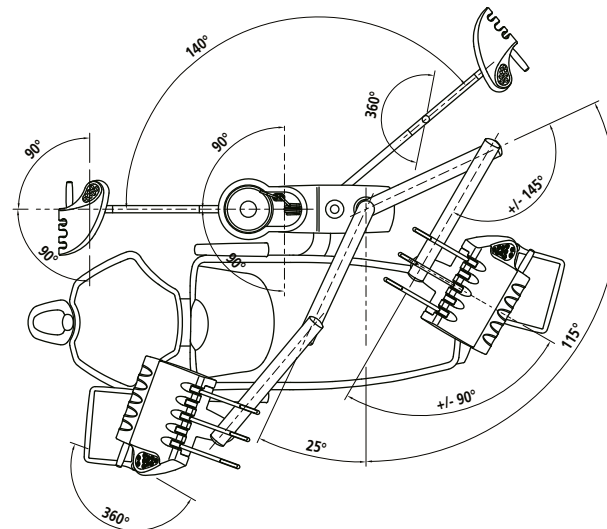Rotazioni possibili • Possible spins • Rotations possibles • Rotaciones posibles • Mögliche Rotationen

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Installation

It is recommended that the unit must be set up allowing a best using for the end user.

Removing the upper and side parts of the packing leaving the assembled chair on the pallet in order that it can be transported better. Removing the protections of the chair and all the fittings contained in the packing itself.

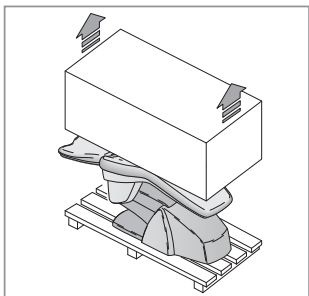
Fig. 1 • Pct. 1 • Abb. 1

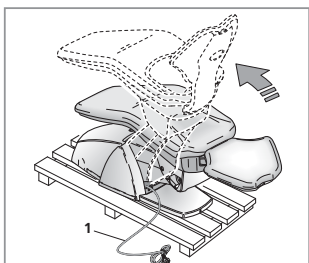
Fig. 2 • Pct. 2 • Abb. 2

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Connecting the feeding cable (1) of the chair, coming out from the base, to a socket. Switch on the main switch and, by the foot control, start the lifting to the maximum height as the backrest movement up to straighten it. After this working disconnect the feeding cable.

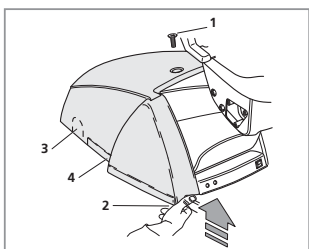
Fig. 3 • Pct. 3 • Abb. 3

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Unscrew the fixing screw (1) of the cover base and remove it pulling up the point (2) upwards for opening it. If the installation can have the entrance of the feedings from the outside, open the entrance (3) already predisposed completely. If the installation can have the assembly of the stability kit, open the predisposed entrance (4) completely.

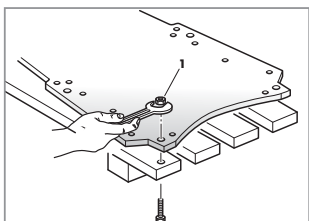
Fig. 4 • Pct. 4 • Abb. 4

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Unscrew the nuts (1) fixing the base to the pallet in order to free the chair from the packaging.

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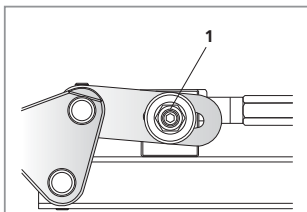
Fig. 19 • Pct. 19 • Abb. 19

Regulations of the articulated arm of dentist's instruments table

After the final installation, the articulated arm could need some regulations:

For accessing to the regulation of the articulated arm, remove the upper cover (1) unscrewing the screws (2).

For regulating the horizontal position of the articulated arm in longitudinal way, loosen the two nuts (1) of blocking to the heads of the hexagonal bar (2). Then turning it right, the profile of the instruments table will be tilted upwards; turning it left, the profile of the table will be sent down.

Do not forget to lock the two blocking nuts again.

Rotating the head of the spring's screw (1) in clockwise sense, the arm will be in position to support a bigger weight. Rotating the head of the spring's screw in anticlockwise sense, the arm will support a smaller weight.

Rotating in clockwise sense the nut (1), the clutch of the vertical movement will be harder. Rotating in anticlockwise sense the nut (1), the vertical movement of the clutch will be soft.

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Place the chair in the pre-arranged point for the installation.

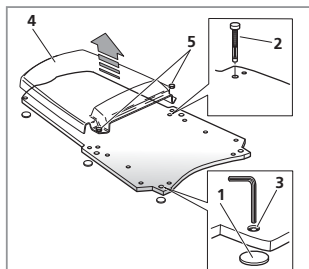
Fig. 6 • Pct. 6 • Abb. 6

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Place the 6 supplied stainless disks (1) under the levelling screws and fixing the base to the ground with the screw anchors (2) according to the kind of material of which the floor is made. Fixing the screw anchors (2) and, if necessary, levelling to the floor the base by the screws (3) taking care that they press on the stainless steel disks. If necessary, working on the rear screws and remove the cover (4), unscrewing the screws (5) and pulling it up.

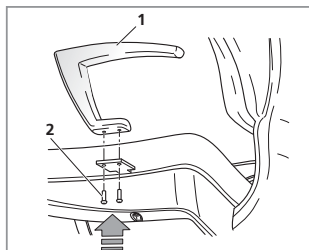
Fig. 7 • Pct. 7 • Abb. 7

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Left arm assembly

Fit the arm (1) on the proper bracket and then tighten completely fixing the screws (2).

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Attention!

For packing reasons, the arm is assembled downwards.

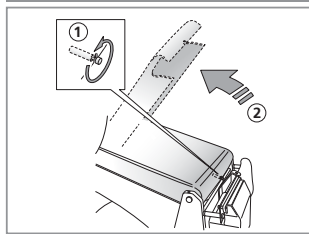
Fig. 8 • Pct. 8 • Abb. 8

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For entering inside the arm of the chair and for passing the water unit connection, loosen the screw (1), pulling up the cover (2) and then ahead until it is completely pulled out taking care of placing the chair at proper height. For resetting up make the reverse operation.

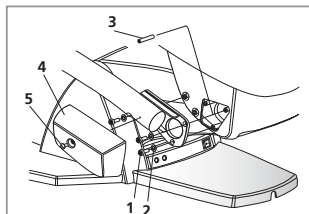
Fig. 9 • Pct. 9 • Abb. 9

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Assembly the water unit of the units on the chair's support through the screws (1) and the washer (2) supplied with the accessories. Pull it levelling to the floor before tightening the screws. The screws (3) must be used as indicator in order to facilitate the assembly. When everything is completely well done, put the base cover (4) and fix it by a screw (5).

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Then connect the alimentation of tubes: air (white colour), water (green colour) and the aspiration tube. Verify that the tubes of the fluid and the liquid are perfectly cleaned up.

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Electrical connections

Electrical connections to the chair's base:

Take away the plastic protection 1 and then connect the cables coming from the water unit to the connectors CN2 and CN5.

Then connect the operating lamp's 2 heads coming from the water unit clamping block CN11 using the usefull contacts for the used lamp.

Then connect the cables coming from the Earth terminal. The cable for the activation command of the aspiration pump must be connected to the clamping block M4 on which there will be a cleaned contact in closing to the raising of the aspiration cannuls will be had.

The open door's cable must be connected to the clamping block M3 on which there will be a cleaned contact in closing will be had when the key board of the open door is activated.

The external circuits of the opening door and of the pump should have a maximum tension of 24 V. Finally the feeding cable should be connected to the entrance clamping block.

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Assembly the arm of the dentist's instrument table of the unit (before passing the cable and the tubes into the hydro group) and to block excursion of the arm by inserting the screw (1) and the nuts (supplied with the accessories) in the hole of the hinge. On the control card of the low tension, connect the cable of the arm into sockets CN14. The tubes are connected to the distributor of the water unit, respecting the colours of the opposite connections.

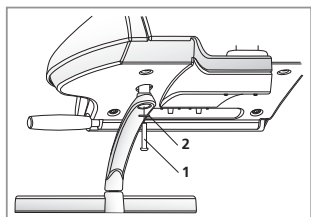
Fig. 11 • Pct. 11 • Abb. 11

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Fixing the tray's support under the dentist instrument table using the given washer (2) and the screw (1).

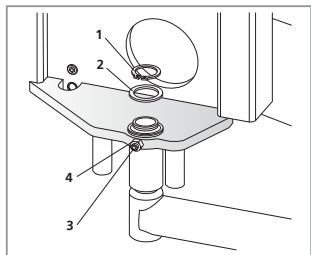
Fig. 12 • Pct. 12 • Abb. 12

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Introduce the cable and the hinge of the assistant's arm into its place (on bottom of the case, chair side) and to block the hinge with the elastic ring Seeger (1) and its reinforced spacer (2).

Pass the cable into the hydro group until grafting it into the sockets CN6 and CN7.

Regulate the friction screw (3) and tighten the blocking lock nut (4).

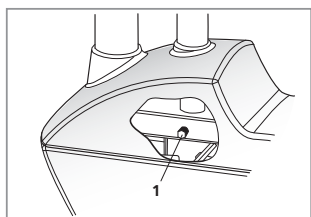
Fig. 13 • Pct. 13 • Abb. 13

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Insert the operating lamp support into the place and block it tightening the screw (1) already positioned inside the hydro group. Pass into the support one guide chain and attack it by an adhesive tape to the outgoing cable from the hinge of the operating lamp's arm; pulling at the same time the guide chain, to lodge the hinge of the support, then connect the clips of the lamp's cable to the socket M1.

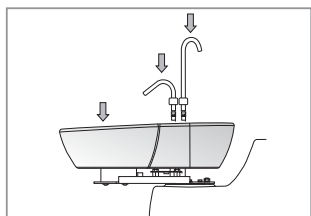
Fig. 14 • Pct. 14 • Abb. 14

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Place the cuspidor bowl in its place. Put in its place the cup filler jets and the bowl flush pipe.

Place the drainage filter and its coverage.

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